
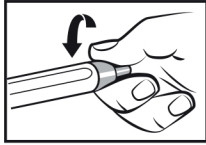
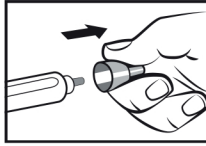
Şekil 1

Şekil 2

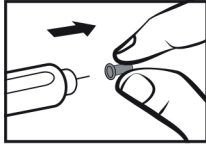Şekil 3

Şekil 4

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## İyon 100 Sterile single use pen needles

Disposable device, pyrogen-free, non-toxic.  
In individual protective packaging.

### Indications

Alvita pen needles are designed for use with a pen injector for the subcutaneous injection of insulin. The needle can be connected by screwing onto the insulin pen.

⚠ Attention, please read the instructions for use

### Warnings and Precautions for use:

- ⊘ Do not use it if package is opened, damaged or trimmed.
- Contains 100 disposable sterile hypodermic needles
- Do not use curved or damaged needles without a protective label.
- Keep out of the reach of children.
- Sterilized with ethylene oxide. Non-toxic and non-pyrogenic.
- Sterility of the needles is guaranteed if the protective label is not damaged.
- Never carry the needle attached to the insulin pen.
- Do not inject the needle through your clothes. Inject it directly onto your skin.
- Can be used in children and adults.
- Do not use if the product has expired.

	Serial number.
	Expiry date.
	Sterilized with ethylene oxide.
	Do not re-sterilize.
	Do not use if the package is damaged.
	Do not re-use.
	Keep away from sunlight.
	Keep dry.
	Non-pyrogenic.
	Latex-free.
	Handle with care.
	Do not use damaged tipped needles.

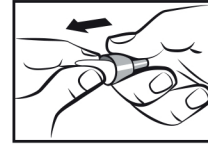
Figure 1

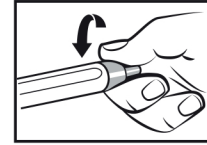
Figure 2

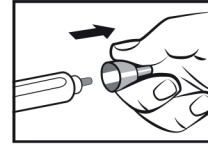
Figure 3

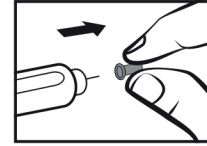
Figure 4

### Instructions for use:

- To be used to inject insulin subcutaneously from the insulin pen.
- Remove the outer protective label on the needle. (Figure 1)
- Push the needle onto the insulin pen and turn clockwise until it is on tight. (Figure 2)
- Pull off the outer protective cap and keep it for later. (Figure 3)
- Pull off the inner protective needle cover and throw it away. (Figure 4)
- Check the insulin flow in the needle before any injection. Insert the needle into the skin as indicated by your doctor and inject slowly.
- Place the outer protective cap back onto the insulin pen.
- Take the needle out safely by turning the protective cap counter clockwise.
- Do not reuse the same needle. Use a new needle for each injection.
- Dispose of in a sharps disposal container.

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